


# 结肠离断实例

## Transection: Colon

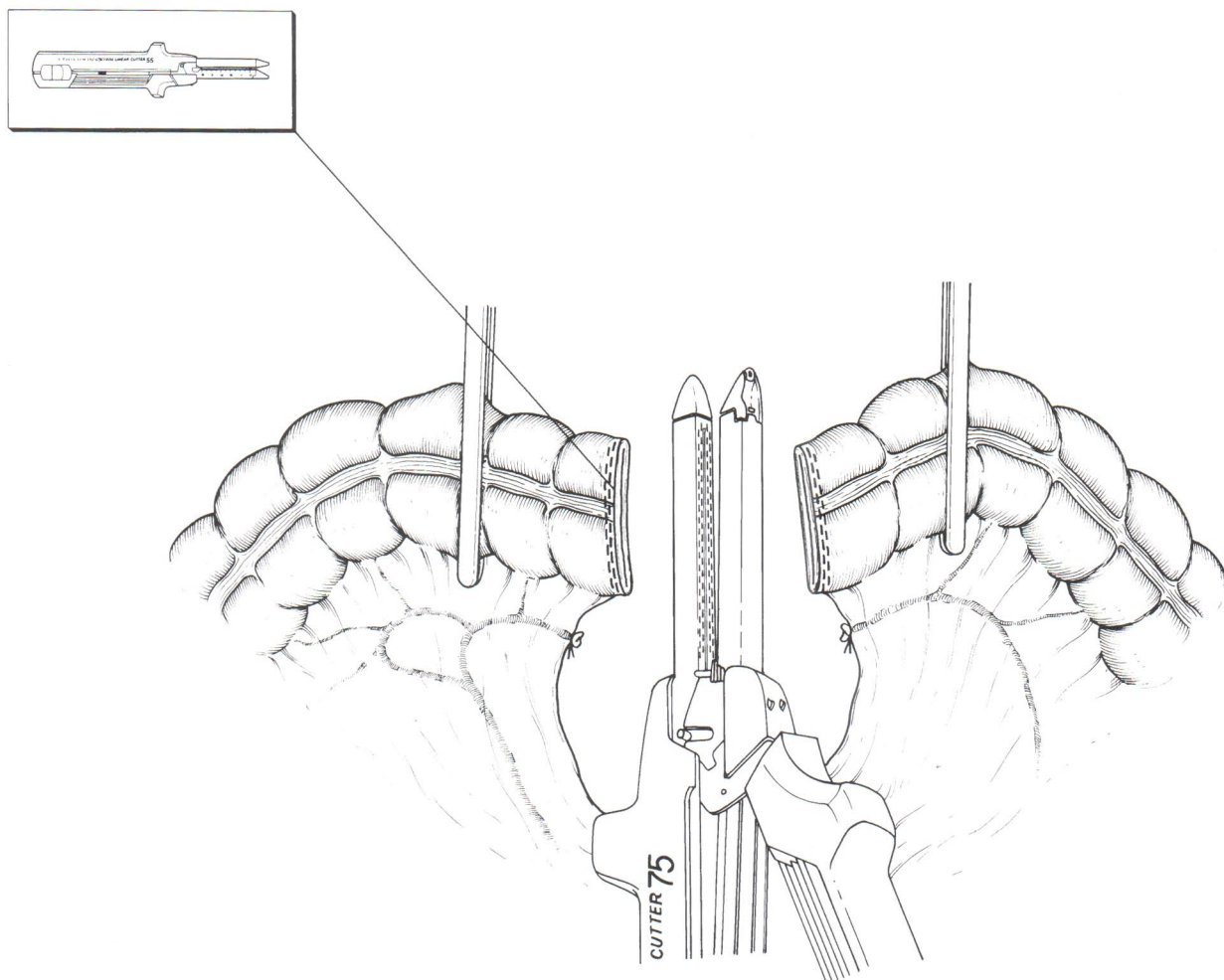

### 1.

处理了肠系膜后，用直线形切割吻合器切断肠管。应注意让叉子间的组织均匀地展开。另外，至叉子上刻度的零点处为能够进行切离的范围。因此，为了使需切离的组织能够进入这一范围，必须调整器械的位置。这时，为了方便起见，可使用器械中间的锁定结构，离断操作完成后，必须确认止血情况。

After the manipulation of its mesentery, the colon is transected with the linear cutter. Ensure that the tissue lies flatly and evenly between the two forks. The transection range is up to the zero mark indicated on the cartridge fork. Therefore, it is necessary to adjust the position of the linear cutter to ensure the tissue that is to be transected is placed within the range. The locking structure at the middle part of the instrument may be used for easy manipulation. Examine the staple lines for hemostasis after the transection.