
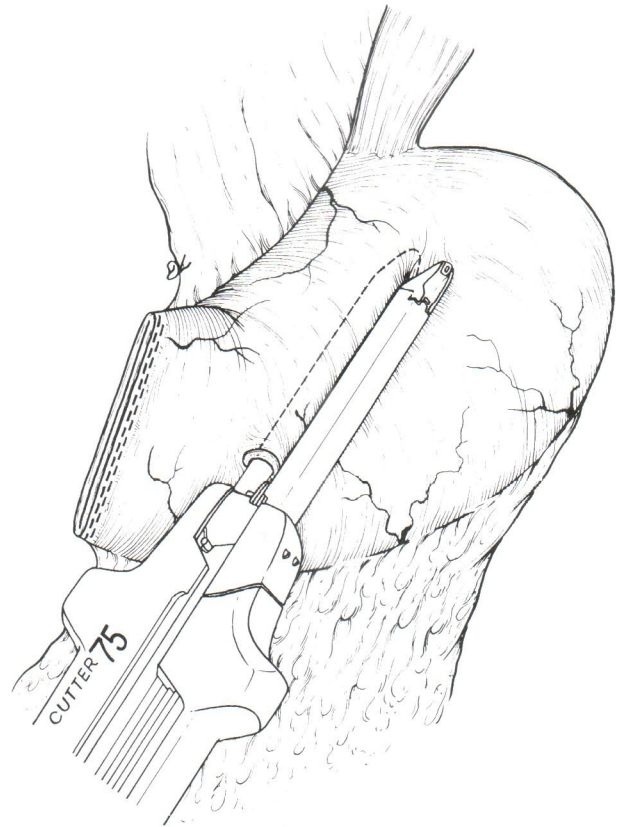


胃切开术实例

Transection: Gastrotomy

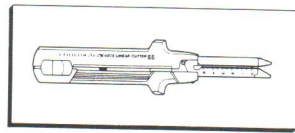

1.

在拟定的切开线始端用手术刀作一 1cm 左右的切口，从这里插入直线形切割吻合器的钉砧叉，切开胃的前壁。切开的长度根据需要而定。切开后应确认止血情况，如果止血不充分，可使用电刀止血，但应注意不使缝钉通电。

A 1 cm incision is made at the start of the target cutting line, through which the anvil fork of the linear cutter is inserted. Then, cut the anterior gastric wall. The length of the incision depends on the need of each particular case. Examine the staple lines for hemostasis when the procedure is completed. Electro-coagulation may be used if hemostasis was not satisfactory achieved. Be cautious because the staples are electricity conductors.

